


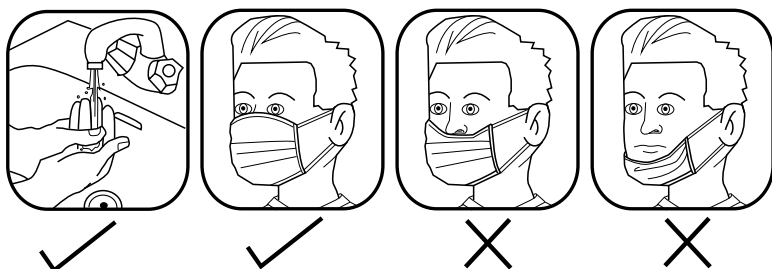
Product reference : TRP117 Disposable Medical Face Mask

REGATTA 
PROFESSIONAL

 These face masks bear CE marking to demonstrate compliance with MDD 93/42/EEC Annex I Health & Safety requirements - EN 14683 : 2019 - Medical Face masks ISO 13485 : 2016 - Quality Management System	Bacteria filtration efficiency	BFE ≥ 95%
	Microbial cleanliness ≤ 30	Breathability < 40
CLASS 1 MEDICAL DEVICE	TYPE I Suitable for use in situations where moderate to light amounts of fluid or aerosols are produced. Please note this product is not classified as PPE	Splash Resistance (not tested) - NOT APPLICABLE TO TYPE 1
These medical face masks are intended to be worn by patients and other persons to reduce the risk of spreading of infections, particularly in epidemic or pandemic situations		- NON STERILE - SINGLE USE ONLY

3 layer structural protection - comfortable & breathable - shaped nose clip - ear loops - fitted to face

Instructions for use:



How to put on the face mask -

- Clean your hands with soap and water or hand sanitizer before touching the mask.
- Remove mask from package, take out the mask, hold by the two ends/ ear loops with the nose bridge at the top - white side should be facing towards the inside to touch the face, blue side should be facing to the outside.
- Cover the nose, mouth and jaw with the mask and loop the ear loops over each ear.
- Adjust the position of the mask, mould or pinch the stiff edge over the bridge of the nose for a comfortable fit.
- The mask should cover the nose, mouth and chin at all times during wearing.
- At no time should the mask hang around the neck of the wearer.

How to remove the face mask -

- Clean your hands with soap and water or hand sanitizer before touching the mask.
- Avoid touching the main part of the mask as this is contaminated, only touch the ear loops.
- Hold both of the ear loops and gently lift and remove the mask.
- Dispose of the mask according to the requirements of the hospital or environmental protection department.
- Clean your hands with soap and water or hand sanitizer.

Warnings & cautions:

- Please read the instructions on how to put on the face mask and how to remove the face mask, carefully before use.
- This product is single use, it should not be re-used.
- In case of liquid splashing, mask damage, moisture and/ or significant increase in respiratory resistance during wear please remove and replace immediately.
- Please check the packaging before use, if the packaging is damaged - do not use.
- Non sterile masks should not be used in areas with strict microbiological index control.

Storage method:

This product should be stored in a cool, dry, corrosive gas free, well ventilated and clean environment.

Manufacturer - Xiamen Hurun Medical Technology Co., Ltd Unit 2, 4th Floor, Building 4, Xiamen Biomedical Industrial Park 2056 Wengjiao West Road Haicang District Xiamen 361026	EU Representative - Luxus Lebenswelt GmbH, Kochstr. 1, 47877 Willich Germany Contact : Info.m@luxuslw.de	Importer - Regatta Ltd, Manchester, United Kingdom. M41 7RR www.regattaprofessional.co.uk Contact : professional@regatta.com Tel: 0161 749 1200
--	--	--